



How to Use
1 Pull the paper tab off the pen needle.

2 Screw needle onto the end of the insulin pen.

3 Remove outer needle cover.

4 Remove inner needle cover to expose the needle.

Your pen is now ready for use.

Medoject Fine® Pen Needles

Medoject Fine Pen Needles are made of polypropylene hub and stainless steel tube. They are intended for Insulin application via most Insulin pens. The needles are sterile and individually packed.

COMFORTABLE INJECTION

European, premium needle quality, triple beveled tip and precision lancet point grinding.

THIN WALL TECHNOLOGY

Ensures optimal insulin flow.

SILICONIZED NEEDLE

Minimizes irritation and gives a smooth penetration.

SHORT NEEDLES

Reduce risk of a painful injection into the muscle.

VERSATILE

Fit most commonly used insulin pens.

COLOUR-CODING

Simple identification of different needle lengths.

Technical Specifications

Colour Code	Specification	Article No.
Yellow	29G x 10 mm	CHPN2910
Pink	29G x 12.7 mm	CHPN29127
Dark blue	30G x 6 mm	CHPN3006
Light blue	30G x 8 mm	CHPN3008
Green	31G x 4 mm	CHPN3104
Dark violet	31G x 5 mm	CHPN3105
Dark blue	31G x 6 mm	CHPN3106
Light blue	31G x 8 mm	CHPN3108
Green	32G x 4 mm	CHPN3204
Dark violet	32G x 5 mm	CHPN3205
Dark blue	32G x 6 mm	CHPN3206
Green	33G x 4 mm	CHPN3304 *

MEDOJECT Fine Pen Needles are available in many sizes. Ask your diabetes educator or pharmacist for advice.
* Not FDA cleared.

Shelf life 5 years

Packaging:
Primary packaging: 100 pcs.
Transit packaging: 2,000 pcs.
Pallet: 160,000 pcs.

