



SAFETY LANCETS

Vitrex® Sterilance Lite III
Sterile Safety lancet

For your safety and patient comfort

BUTTON ACTIVATION.

USER SAFETY needle is fully shielded before and after use.

USER CONVENIENCE pre-loaded, pressure activated design.

PATIENT COMFORT high quality, ultra sharp tri-bevel needle or bi-bevel blade, high speed penetration.

EFFICACY variety of point configurations and penetration depths to suit most capillary blood requirements.

APPLICATION blood glucose and other in vitro diagnostic tests.

PATIENT SAFETY sterile, single-use, unable to be re-used.



Recommended use

- 1. Twist off the protective cap.
- 2. Select fingertip insertion site slightly off centre to minimise any discomfort and maximise blood flow. Hold the lancet firmly against site and press the release button to activate the device.
- 3. Gently massage the finger in the direction of the puncture site to obtain the required volume of blood.
- 4. Dispose of safety lancet in a suitable safety container.

WARNING: Do not use if protective cap has previously been removed.

Technical Specs.

Colour Code		Configuration	Penetration Depth	Article No.	Application
Green	●	18G blade	1.8 mm	373213	Special baby care and maternity units for heel puncture
Orange	●	21G needle	1.8 mm	377213	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests. For paediatrics and most normal adult skin
Pink	●	21G needle	2.4 mm	378213	As above but for thicker adult skin
Yellow	●	26G needle	1.8 mm	375213	Diabetes clinics/hospital wards for blood glucose monitoring
Blue	●	26G needle	2.4 mm	376213	As above but for thicker adult skin
Purple	●	28G needle	1.8 mm	374213	Diabetes clinics/hospital wards for blood glucose monitoring

Shelf life 5 years

Packaging:

Primary packaging: 100 pcs.
Transit packaging: 1,000 pcs.
Pallet: 90,000 pcs.



Transit packaging

Pallet