



SAFETY LANCETS

Vitrex® Sterilance Press II  
Sterile Safety lancet

Health workers worldwide face accidental needle-stick injury while caring for patients, putting themselves at risk to bloodborne illnesses including the AIDS virus and Hepatitis.

**PRESSURE ACTIVATION** ensures a good blood sample every time.

**USER SAFETY** needle is fully shielded before and after use.

**USER CONVENIENCE** pre-loaded, pressure activated design.

**PATIENT COMFORT** high quality, ultra sharp tri-bevel needle or bi-bevel blade, high speed penetration.

**EFFICACY** variety of point configurations and penetration depths to suit most capillary blood requirements.

**APPLICATION** blood glucose and other in vitro diagnostic tests.

**PATIENT SAFETY** sterile, single-use, unable to be re-used.



Recommended use

1. Carefully twist off the protective cap.
2. Select site and press the device firmly against the site to release the lancet.
3. Dispose of safety lancet in a suitable safety container.

**WARNING:** Do not use if protective safety cap has previously been removed.

Technical Specs.

Colour Code		Configuration	Penetration Depth	Article No.	Application
Green	●	18G blade	1.8 mm	393213	Special baby care and maternity units for heel puncture
Blue	●	21G needle	1.8 mm	397213	As Yellow but for increased blood flow
Orange	●	21G needle	2.2 mm	395213	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests
Pink	●	21G needle	2.8 mm	396213	As Orange but for thicker adult skin
Yellow	●	26G needle	1.8 mm	394213	Diabetes clinics/hospital wards for blood glucose monitoring
Purple	●	28G needle	1.8 mm	398213	Low blood flow, less pain
Turquoise	●	30G needle	1.8 mm	399213	Low blood flow, less pain

Shelf life 5 years



Packaging  
Primary packaging: 100 pcs.  
Transit packaging: 1,000 pcs.  
Pallet: 98,000 pcs.